



TRISAN AUSTRALIA PTY LTD

ABN: 58 008 270 040
 32-34 King William Street
 KENT TOWN SA 5067
 Tel: +61 8 8363 1770
 Fax: +61 8 8362 8070
 Email: sales@trisan.com.au



GENERAL INFORMATION and USER HINTS

HOME FIRE SAFETY IS IMPORTANT !!!!!!!

This system is designed to help awaken people with a severe to profound hearing loss in case of fire.

About the VIBRACON System:-

The Vibracon **Smoke Alarm Monitor (SAM)** provides the user the freedom to relocate the monitor anywhere in the home without the need to lay wiring directly from the smoke alarm. This can offer advantages in terms of cost and convenience.

The **Smoke Alarm Monitor (SAM)** receives a radio signal from the smoke alarm, having a range of up to 40 meters, provided aerial is fully extended. The Monitor receives the signal and triggers a bright flashing strobe light. In addition it has two output connectors to which maybe attached a vibrating Bed Shaker disc, or other attention-seeking device.

The **Smoke Alarm** (smoke detector) runs off a long-life Lithium battery, which should last up to ten years, depending upon usage, and is best located on the ceiling outside the bedroom door. If the Smoke Alarm is activated it makes the normal warning sound regardless whether SAM is switched on/off.

The VIBRACON Smoke Alarm Monitor (SAM) is normally located in the bedroom, on a bedside table. The under-pillow Bed Shaker disc is placed inside the pillow slip under the pillow to help awaken the person when an alarm is activated.

QUICK START WITH VIBRACON SAM UNIT

- Step One:** Plug Vibracon SAM unit into mains power. Switch power ON.
 Red light will flash. The unit is now in STANDBY mode.
 Charge SAM for 24 hours before use.
- Step Two:** Slide Switch on back of unit to ON position.
 Red light goes off.
 Green light comes on.
- Step Three:** Turn aerial wire on back from horizontal to vertical position for best transmitting reception.
- Step Four:** Plug Under-Pillow Bed Shake Disc into back of unit.
- Step Five:** Push button on Test Fob to test unit. Strobe light will flash and Bed Shaker disc will vibrate if unit is operating correctly.
- Step Six:** ***Read instructions on installation of Smoke Alarm First.***
 Press and hold the Test Button on Smoke Alarm until the alarm sounds (it may take up to 10 seconds).
Figure 7 on page 8 of Smoke Alarm instructions.
 The alarm will stop sounding shortly after the button is released. If the alarm fails to sound, follow checklist instructions.
 This procedure will ensure Smoke Alarm is activating strobe and under-pillow bed shake before installation.
- Step Seven:** Repeat Steps Five and Six once each month.

For discontinued use e.g. in the case of unit being sent back to factory for repair or recycle, **slide switch on SAM to "OFF"** position and remove from mains power.